Strada Regina
CH-6874 Castel San Pietro - Switzerland
Phone: +41 91 696 60 60 Fax +41 91 696 60 66
Info@medacta.ch www.medacta.com

KNEE SPORTS MEDICINE
GINOCCHIO SPORTS MEDICINE
KNEE SPORTS MEDICINE
KNEE SPORTS MEDICINE
JOELHO SPORTS MEDICINE
GENOU SPORTS MEDICINE
RONATO SPORTS MEDICINE

CAUTION
For U.S. specific instructions for use, please go to www.medacta.com or call toll free 800-901-7836.

SYMBOLS / SIMBOLOS / PICTOGRAMMES / PICTOGRAMME / PICTOGRAFES / PICTOGRAMAS / PICTOGRAMMAS / EIKONOSYMBOLA

Do not reuse / Non riutilizzare / Niet hergebruiken / Nicht wiederverwenden / No reutilizable / Não reutilizar / Ne pas réutiliser / Μην επαναχρησιστείτε

Do not resterilize / Non ristérilizzare / Bitte nicht re-sterilisieren / No resterilizar / Não reestérilizar / Ne pas restériliser / Μην επαναχρησιστείτε λύματα

Caution, read the accompanying documents / Attenzione, consultare i documenti di accompagnamento / Opelet, raadpleeg de bijgeleverde documenten / Achtung, Beigefügte Dokumente beachten / Atención, consulte los documentos que se acompañan / Αντέξτε τα συνοδευτικά έγγραφα

Consult instructions for use / Consultare le istruzioni di utilizzo / Gebruiksaanwijzing / Bitte lesen Sie die Gebrauchsanweisungen / Consultar las instrucciones antes de su uso / Consultez les instructions de usage / Consultez les instructions d'utilisation / Ολόγραφη χρήσης

Do not expose to sunlight / Non esporre alla luce del sole / Niet blootstellen aan zonlicht / Vor Sonnenlicht schützen / Μην ξεθέτετε σε ήλιακο φως

Store in a dry place / Conservare in luogo asciutto / Droog bewaren / Trocken aufbewahren / Conservar en lugar seco / Conservar em local seco / Conservar au sec / Φυλακτέστε σε ξηρό χώρο

Do not use if package is damaged / Non utilizzare se la confezione è danneggiata / Niet gebruiken als verpakking beschadigt is / Bitte nicht verwenden, falls Verpackung beschädigt ist / No utilizar si el embalaje está dañado / Não usar se o embalagem estiver violada / Ne pas utiliser si l'emballage est abîmé / Μην χρησιμοποιείτε εάν η συσκευασία είναι καταστραμμένη

Use by / Utilizzare entro il / Te gebruiken voor / Zu verwenden bis / Utilizar antes de / A utilizar antes de / A utiliser avant / Ημερομηνία λήξης

Lot number / Numero di lotto / Nummer van de partij / Waren-Lot-Nummer / Número de lote / Número de lote / Número de lot / Λημάνιο παρτίδας

Reference number / Référence commerciale / Handelsreferenz / Referenznummer / Número de Referencia / Référence commerciale / Référence commerciale / Νομήριο παραπομπής

STERILIZED BY IRRADIATION / Sterilizzato mediante irradiazione / Gesteriliseerd door stroosaling / Sterilisiert durch Strahlung / Esterilizado por irradiación / Stérilisé par irradiation / Αποτριπτικό με ακτινοθεραπεία

ENGLISH HIP ACCESS KIT - INSTRUCTION FOR USE - ACCESS KIT

Important notice: the device(s) can be prescribed and implanted only by a medical doctor legally authorized to perform this type of surgery.

GENERAL
Use of the instrumentation requires knowledge of the anatomy, biomechanics and reconstructive surgery of the locomotive system. The instrumentation can be used only by a qualified surgeon who practices with an awareness of current advances in science and surgical techniques. The user should ensure that the instruments are undamaged and in good working order before use. The user should also take all necessary precautions to avoid accidents (gloves, protective glasses, etc.).

PRODUCT DESCRIPTION
Medacta Hip Access Kit is provided sterile and contains sterile single-use disposable instruments. The intended performance of each component of Medacta Hip Access Kit is listed below:

- Guidewire: ensure the the artroscopic portals remain open, after removing the needle
- Blades: enlarge the access into the hip capsule by cutting the tissue
- Access Needle: punctures the skin and tissues of the patient at the surgical site

Medacta Hip Access Kit (Ref. 05.11.10.0041) instruments:

ITEM	REF.NO.	DESCRIPTION	QTY	MATERIALS
1	66.10.078	AK Nitrod Guidewire	2	Nitrod
2	66.10.079	AK Blade Straight	1	AISI30
3	66.10.080	AK Blade Curved	1	AISI30
4	66.10.081	AK Access Needle	2	AISI304, ABS, Polycarbonate

INTENDED USE
Medacta Hip Access Kit is designed to provide the surgeon with all the main instruments to gain access to the surgical site during arthroscopy procedures.

Possible Adverse Effects:

- Nerve damage
- Infection following the procedure
- Introduction of foreign materials can result in an inflammatory response or allergic reaction

WARNING
Check the expiration date prior to use. Hip Access Kit instruments are meant to be single use. Reutilizing them is strictly forbidden and would lead to unpredictable outcomes.

Proper hip position and distraction should be confirmed prior to advancement of distension needle, guidewires, switching stick and cannulae. Neurovascular structures must be avoided when advancing distension needle, guidewires, switching stick and cannulae.

This kit contains instruments that are very sharp. Do not touch sharp areas with fingers. Injury may result. Dispose according to hospital guidelines pertaining to sharp disposable instruments.

Visually inspect the instruments after use in order to verify they did not experience any mechanical damage which may cause release of particles in the human body. Any non-functional instrument should be immediately returned to Medacta along with its label. The type of malfunction should be reported.


PRECAUTIONS
The instruments in this kit are designed for use by surgeons experienced in the appropriate specialized procedures.

It is responsibility of the surgeon to become familiar with the proper technique.

INSTRUCTIONS FOR USE

- Remove items from the sterile package using aseptic technique.
- Access needle: designed with a cannulation suitable to receive a 1.50 mm. guidewire end with a working length of 180 mm. Access needle is used to get entrance to the joint according to the desired/preferred portal position.

WARNING
To reduce the risk of damage to the femoral head during the insertion phase, an arrow is marked on the plastic connection indicating the opening face of the needle.

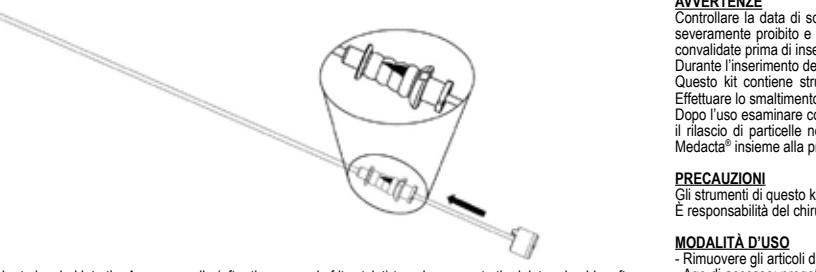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Guidewire: flexible wire to be sled into the Access needle (after the removal of its stylet) to gain access to the joint and guide, after the removal of the access needle, the insertion of other instruments (like switching stick and cannulae) with a working length of 430 mm.

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