# 44 ectaFlip INTRA ARTICULAR EXPANDER Surgical Technique Sports Med **⊿**edacta •



# INDEX

1.	INSTRUMENTS OVERVIEW	2
	1.1 MectaFlip description	
	1.2 Surgical steps	_

## 1. INSTRUMENTS OVERVIEW

The Medacta MectaFlip is a disposable instrument for Hip arthroscopy soft tissue management designed to retract the soft tissue from the surgical site and to increase the visibility on the intra-operative space during hip arthroscopic procedures.

### 1.1 MECTAFLIP DESCRIPTION

The Medacta MectaFlip Intra Articular Expander is a Ø3 mm needle with a patented mechanism allowing for flipping the tip by 90°. The plastic barrel control the movement of the flipping tip used to retract the soft tissue in a precise way thanks to the dedicated mechanism. The detachable plastic counterplate is designed to compress a wider area of soft tissues when coupled with the ratchet of the main body, allowing the use of multiple devices at the same time if bigger retraction is needed.



3 lenghts available:

- 05.09.10.0001 MectaFlip 80mm
- 05.09.10.0002 MectaFlip 120mm
- 05.09.10.0003 MectaFlip 150mm
- Movable tip (0°-90°) controlled by the plastic barrel
- Plastic counterplate that engages with body ratchet

### 1.2 SURGICAL STEPS

Insert the main body through the skin until the desired position is reached, keeping the tip in straight configuration.













Flip the distal tip by moving the plastic barrel until the final position is reached

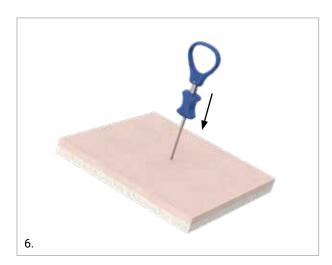



Couple the star counter plate over the main body pressing the star handles

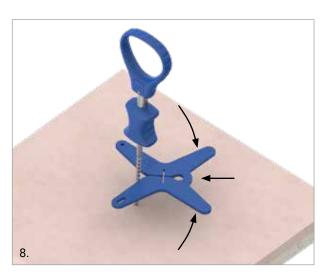







Press the counterplate towards the skin while pulling the ring handle with the other hand in order to compress and raise the soft tissues





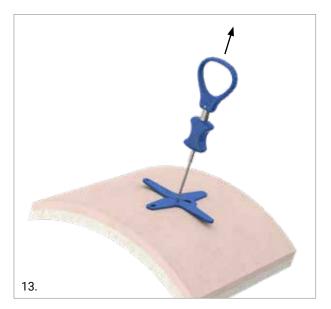

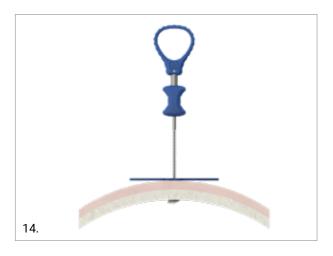

Multiple devices can be used at the same time if more visibility is needed





Part numbers subject to change.

# NOTE FOR STERILISATION

Medacta Mectaflip is delivered sterile.





### Medacta International SA

Strada Regina - 6874 Castel San Pietro - Switzerland Phone +41 91 696 60 60 - Fax +41 91 696 60 66 info@medacta.ch

Find your local dealer at: medacta.com/locations

All trademarks and registered trademarks are the property of their respective owners. This document is not intended for the US market. Please verify approval of the devices described in this document with your local Medacta representative.

MectaFlip Surgical Technique

ref: 99.111SMH.12 rev. 01

Last update: March 2019 **C € 0476**